

Safety recommendations


- Read the instructions for use. (www.lopital.com).
- Maximum load 227 kg / 500 lbs.
- Make sure that the Marina basic / Marina deluxe is on the brake during client transfers from and to the shower trolley.



Electric manual control

Up		
Down		
Backrest up (deluxe)		
Backrest down (deluxe)		

Hydraulic control

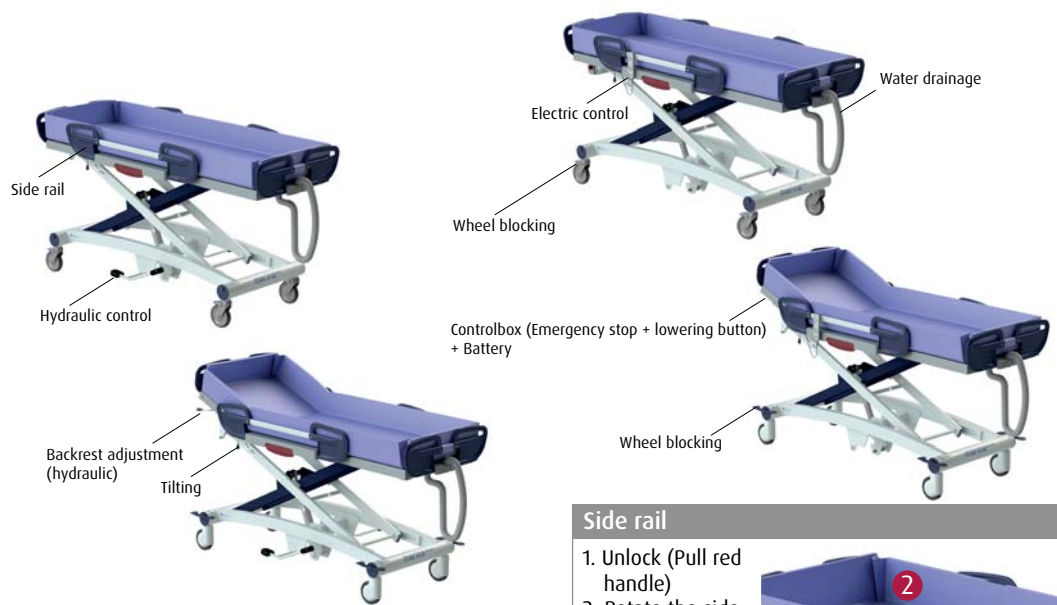

Tilting (Anti-Trendelenburg)

Wheel blocking Marina basic (61002000/2050)

1. Wheel blocking
2. Unlocking

Wheelblocking Marina deluxe (61002010/2060)

1. Brakes applied (inside lever down)
2. Forward movement (Footend) (outside lever down)
- Horizontal setting : Brakes release

Side rail

1. Unlock (Pull red handle)
2. Rotate the side rail to the right position.

Water drainage

Water is drained away from the Marina basic / Marina deluxe via a drainage hose. To do this, remove the hose from the recess (1) and route it to the lowest possible drainage point. Always click the hose into the recess of the lying plate during transport.

Charging battery (electric)

Charging the battery before it is completely empty will extend the lifespan.

Battery indication on hand control.

	Fully charged		1/3 capacity, Charging advised
	2/3 capacity		charging required, an acoustic signal sounds

Emergency stop & lowering button (electric)

1. Operate the emergency stop in an emergency, if the upward or downward movement must be stopped immediately.
2. By pressing the lowering button in an emergency, the shower trolley will return to the lowest position.

Cleaning and maintenance

The Marina basic / Marina deluxe must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the mattress can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use. (www.lopital.com)

According to the regulation for medical devices, a trolley must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

In the event of a breakdown, contact your Lopital distributor.

Instructions for use

For the full Instructions for use go to www.lopital.com

Important!
This quick user manual may not be considered a replacement for the instructions for use.